Actimo Delta

DE

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Actimo Delta

UK

Carefully read the whole of the instruction manual before you assemble, adjust or use the Actimo Delta for the first time.

Intended purpose

As a walking frame for the aiding of the walking of people who are restricted in their movements and mobility. Tray, basket and bag for the transport of utensils and everyday objects.

Indication

The Actimo Delta can be used in cases of slight balance disorders and limited mobility, due to loss of strength or reduced mobility. To maintain of physical activity, the Actimo Delta is suitable for indoor and outdoor use.

Assembly / Instructions: Your specialist dealer or care person will willingly help you with the assembly

1. Before unpacking, check the packaging for damage. If the contents are damaged, contact your specialist dealer immediately. The Actimo Delta is delivered in the ready-to-use condition, assembled by the specialist dealer or your care staff and the function checked.
2. The technician / dealer adjusts the handle height to suit the user.
3. Place the Actimo Delta in the direction of travel, unfold and lock it. Ensure that the lock is firmly seated in order to prevent a collapse.
4. For unfolding, place the walking frame in the direction of travel. Hold the walking frame by the handles and pull them apart. Should this be a little stiff, press the frame apart at the running wheels. Now push the quick lock on the folding mechanism to the end stop until it locks into place.
5. For folding together, place the Actimo Delta with the brakes applied in the direction of travel. Remove all accessories. Firmly hold the walking frame by the handle with one hand and pull up the quick lock on the folding mechanism with the other. Should this be stiff, press the frame together at the running wheels.

6. To assemble the handles, remove the black protective caps from the ends of the frame bars. Guide the handle tubes into the openings. The handles are fixed in place by means of the locking levers and can be adjusted to the right height for the user. The locking levers must point outwards.

Safety Instructions

1. When setting the height, the handle bars must only be pulled out to the edge of the rough surface.
2. There is the risk of the trapping of fingers, hands and / or clothing during folding together. Folded together, the Actimo Delta is no longer stable.
3. If the height has not been suitably adjusted, place the Actimo Delta in the direction of travel and apply the brakes. Loosen the fixing screws, set the correct height and re-tighten the fixing screws.
4. A walking frame is not a wheelchair, but exclusively a walking aid.
5. The Actimo Delta is intended for use on level surfaces only. The maximum user weight is 120 kg.
6. During use on inclined surfaces, the rolling speed can be higher than the walking speed. If the Actimo Delta leans too far to the side, there is the risk of tipping over (tilt angle > 5 °). If the resting on the handle bars is too heavy it can lead to the walking frame tipping backwards.
7. Check the brakes before each use!

Maintenance

The Actimo Delta is maintenance-free. If a repair and / or service is necessary, please contact your specialist dealer.

The walking frame can be folded for parking. Before use it must be unfolded again.

Faulty Operation: 

The effectiveness of the brakes can be adversely affected by wear on the wheels. If you notice a change in the braking effect, contact your specialist dealer.

Brakes

Two handbrakes for the rear wheels (b) with Bowden cables (a) and brake callipers. To set the parking brake, simultaneously press the two brake levers downwards until they audibly click into place. They can be unlocked again by briefly pulling them upwards. Always apply the two brakes at the same time. The brakes are set after delivery. For technical assistance, please contact your local specialist dealer.

Scope of Supply


Actimo Delta

1 Basket, 1 Tray, 1 Bag, 1 Instruction Manual

Protect the walking frame from moisture and place it in a safe place to protect the linkage and wheels from damage!

Technical Data / Dimensions and Weight

- Width	62 cm
- Length	58 cm
- Folded Size	24 cm
- Height	80 – 92 cm
- Weight	7 kg

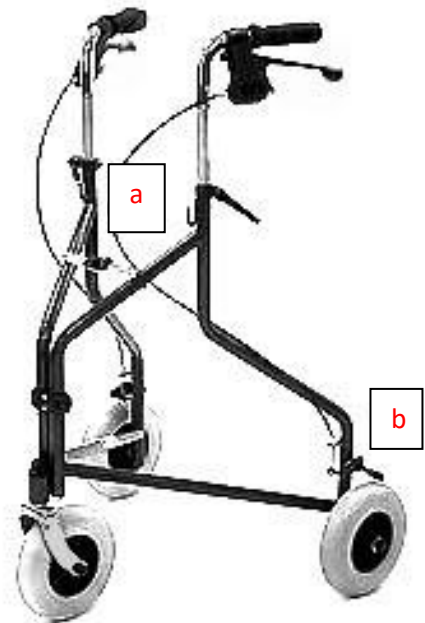- Wheels 19 cm
- Max. Load 120 kg
- Max. weight for basket, tray or bag 5 kg

Cleaning

The walking frame should be cleaned and / or disinfected at regular intervals (absolutely necessary on change of user). Use a mild soap solution without added scouring agents and a soft cloth. Disinfection can be carried out using a commercially available agent in accordance with the manufacturer's instructions, for example, with Descosept AF. Do not use any abrasive or corrosive cleaning agents or high pressure or steam cleaners!

Ensure that no dirt deposits accumulate on the tyres or brakes. These can be removed with a soft brush and warm water.

Contraindications

The use of the Actimo Delta is not advisable for people with

- Perceptual disorders, limited eyesight
- Severe balance problems
- People with amputated limbs, joint damage

Disposal

A worn-out walking frame represents a valuable source of material which can be fed back into the industrial cycle. Therefore do not throw the walking frame to be disposed of into the household rubbish but have it professionally disposed of by an officially approved disposal centre. Your local refuse collection department will be more than willing to give you the appropriate information.

Guarantee

The guarantee is for 24 months and covers all product faults which can be verifiably traced back to manufacturing faults or material defects.

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